

Procedural Reference Guide

NOTE: This is not a substitute for the ReSure Sealant Instructions for Use. For complete instructions including indication for use, warnings, and precautions, please refer to the ReSure Sealant Instructions for Use.



Meticulously dry incision and surrounding tissue (minimum 5 mm radius).



Add two drops of diluent to the blue deposit.



Tilt tray and rapidly mix components in the well with the applicator for up to 5 seconds.



Scoop the material with the applicator to gather an adequate amount to cover the incision.



Ensure incision is dry prior to application. Dry again if necessary.



Apply material over entire incision with good margins.

Ordering Information

ReSure Sealant Kit (10 units) | Product Number: RS-1004-US-10
Customer service: 877-628-8998 | sales@ocutx.com | www.ReSureSealant.com

